


4100-341 Snap Clip is required to allow for the attachment of accessories.

(4100-341 includes a Snap Clip, Cable and Allen Wrenches)

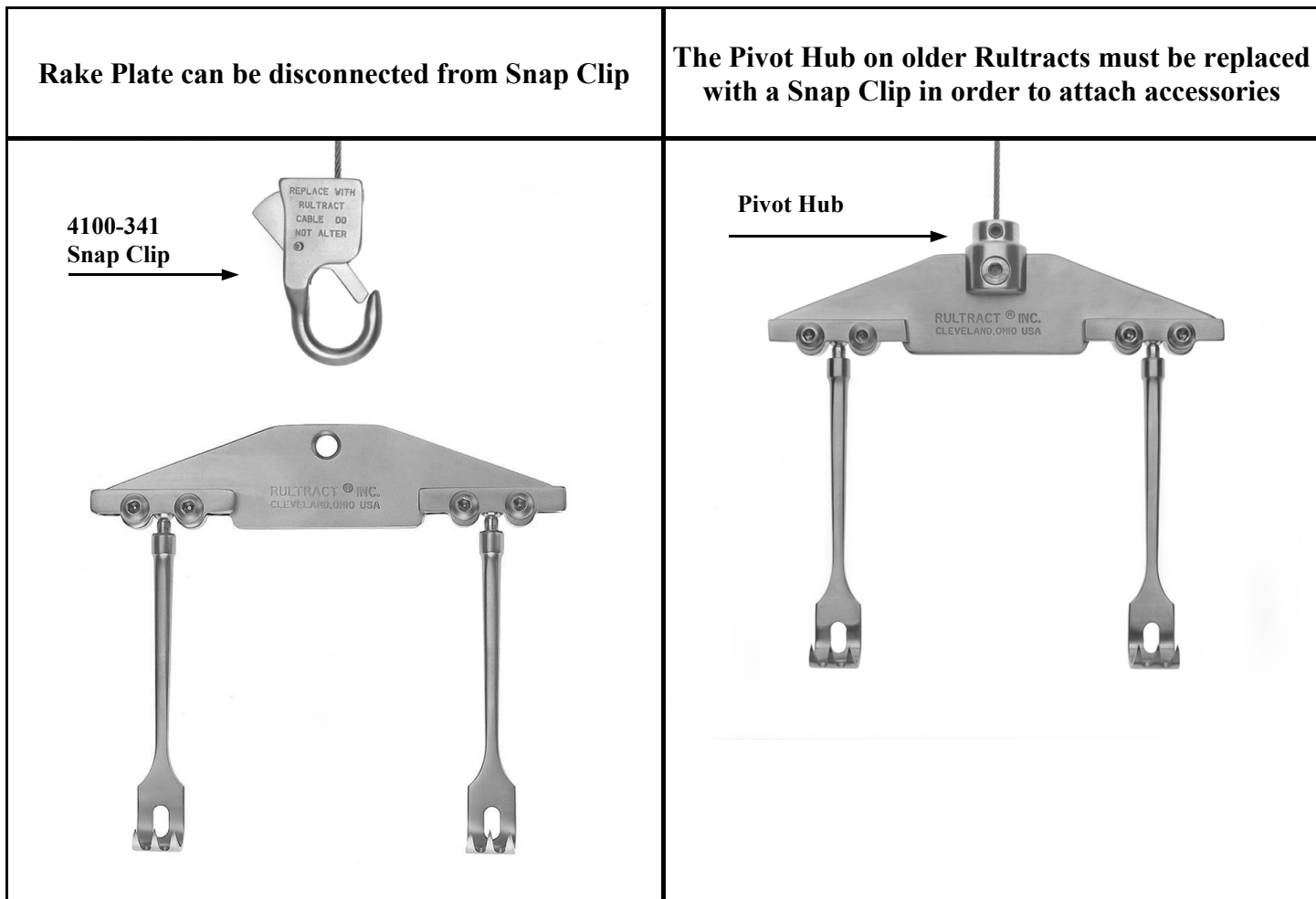

Rultract Inc. 5663 Brecksville Rd. Cleveland, Ohio 44131 U.S.A.
Website: www.rultract.net Email: rultract@aol.com
Phone: 888-550-5695 / 440-376-2077 / 330-856-9808



4100-341

How Supplied: Snap Clip (allen wrenches included)



Rultract Inc. 5663 Brecksville Rd. Cleveland, Ohio 44131 U.S.A.
Website: www.rultract.net Email: rultract@aol.com
Phone: 888-550-5695 / 440-376-2077 / 330-856-9808





5663 Brecksville Rd.
Cleveland, Ohio 44131 U.S.A.
Phone: 888-550-5695 / 440-376-2077 / 330-856-9808
Website: www.rultract.net Email: rultract@aol.com

NOTICE

Rultract[®] is the ONLY authorized service center in the U.S.A.

Any modification or alteration by user will void all warranties and release Rultract[®] from any liability. It is recommended the device be refurbished every 12-18 months. Return it to the above address for service.

WARNING:

Cables worn or damaged through mishandling must be replaced with a **new Rultract[®] cable**. Do not attempt to repair, cut, alter or modify the cable in any way. Cutting and reattaching the cable will cause it to weaken and fray. This will result in failure during use.

CAUTION:

Always allow rake plate to hang freely when winding cable. The weight of rake plate allows for proper alignment of cable on spool. Failure to allow the rake plate to hang freely may cause the cable to kink during winding. Leave approximately one to two inches of cable exposed to prevent flexing of cable on cable guide at insertion site.

Rultract Inc. 5663 Brecksville Rd. Cleveland, Ohio 44131 U.S.A.
Website: www.rultract.net Email: rultract@aol.com
Phone: 888-550-5695 / 440-376-2077 / 330-856-9808

